

SURU INTERNATIONAL PVT. LTD.

## SKIN STAPLER & STAPLE REMOVER



Item Code: 1018,1019

## Features :

- Superior design : Uniform tension for optimum approximation, minimizes tissue compression and trauma
- Superior cosmetic results : Allows good blood supply to tissues, promotes faster and scar free healing
- Sterile stainless steel extractor : For painless and effortless extraction
- Reliable stapling : Good grip, better control, no jamming
- Sterile product for single use




## Easy to Use :

Approximate skin edges in an evented manner using tissue approximation forceps. Place SURUSTAP<sup>®</sup> stapler gently over the center of the approximated skin edges

Squeeze the lever completely till the staple is securely place

(For better results ensure bottom handle completely folds into the top handle)

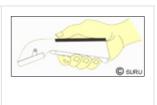

Superior inert stapler pins : Highly resistant to corrosion and most suitable material for Surgical implants



Available Configuration:

• Size available : 35 W / 55 W (7.0 x 4.0 mm)

http://www.suru.com/Surgery-Wound-Care/Skin-Stapler-Staple-Remover

